Scan for more information

Check out our learning platform Reliance U! www.RelianceU.tech

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EMERGO EUROPE
Prinssengracht 20, 2514 AP The Hague, The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional. **NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

RELY-A-BOND® NO MIX ORTHODONTIC BONDING SYSTEM

INSTRUCTIONS FOR USE

EN - ENGLISH
INDICATIONS FOR USE: Rely-a-Bond® is intended for use in bonding metal, composite and ceramic orthodontic brackets. **CONTRAINDICATION:** A No Mix System is not recommended for bonding lingual retainers without bonding pads. **INSTRUCTIONS FOR USE:** **PROPHYLAXIS:** Prophyl the enamel with 1st & Final® pumice. Rinse and dry. **ETCH:** Apply etchant to the enamel. Allow 30 seconds for etching. Rinse and dry **BONDING PROCEDURE:** Apply a thin coat of primer to each etched and dried tooth and the underside of each bracket base. With a spatula, apply a small quantity of adhesive paste to the underside of the bracket base and place on tooth. Within 25 seconds, position the bracket to desired angulation and press firmly. Excess adhesive can be easily removed from the periphery of the bracket base after all of the brackets are in place. An active archwire can be inserted 5 minutes after bracket placement.

② **Special Safety Note:** Brush applicators and mixing pads are single use devices. It is required that a new brush applicator and mixing pad sheet are used for each patient. Spatulas and mixing wells can be autoclaved.
④ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.
④ Etchants and Primer are Corrosive
⚠️**TEMPERATURE LIMITATION:** Store at room temperature (15-30°C). Protect from excessive heat.
CONTENTS OF KIT: 14g Adhesive Paste (jar, syringe tips or push syringes), 16g Primer, 9gm Liquid Etchant, Accessory Pack.

TO REORDER: #1RPP, #1RPF, #RJ, #RPR, #RJF, #RPRF, #RP, #RPT, #RPPT, #RT, #RJT, #RTF, #EL9

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional. **NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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